

Skeletal Dynamics Proximal Humerus Fixation System

INSTRUCTIONS FOR USE FOR THE UNITED STATES

Basic UDI: 00841506138398

R: For use by physicians only. Caution: Federal Law restricts this device to sale by or on the order of a physician.

Failure to follow instructions may lead to patient injury.

This package insert is designed to provide Instructions for Use of the Proximal Humerus Fixation System. Prior to use of the Proximal Humerus Fixation System, the surgeon should become familiar with all information contained in this pamphlet.

Description:

The Skeletal Dynamics Proximal Humerus Fixation system contains bone plates for the repair of the Proximal Humerus. Included in the system are titanium bone plates, screws, and specialized instrumentation.

The Skeletal Dynamics Proximal Humerus Fixation System is comprised of:

- Titanium alloy plates and screws
- Cobalt chrome screws
- Stainless steel k-wires (for provisional fixation, not for implantation)
- System specific instrumentation.

The system is provided non-sterile and is sterilized in the user facility.

Indications for Use:

The Skeletal Dynamics Proximal Humerus Fixation System is indicated for fractures and fracture dislocations, osteotomies, and non-unions of the proximal humerus.

Contraindications:

Prior to using the Proximal Humerus Fixation System, ensure that none of the following patient conditions are present: active or latent infection, insufficient quantity or quality of bone and/or soft tissue, material sensitivity (if sensitivity is suspected, tests are performed prior to implantation), or patients who are unwilling or incapable of following post operative care instructions. The system should not be used in pediatric patients or patients with open growth plates.

General Warnings and Precautions

- The information in this document should be shared with the patient.
- The patient should be informed about the importance of following the post-operative rehabilitation prescribed to fully understand the limitations in activities of daily living. The patient must be warned that failure to follow postoperative care instructions may cause the implant or treatment to fail.
- The patient must be cautioned about the use, limitations, and potential adverse effects of this device including the possibility of delayed union, non-union, device or treatment failure as a result of loosening, stress, excessive activity, or weight bearing or load bearing, and the possibility of nerve or soft tissue damage related to either surgical trauma or the presence of the device.
- Potential construct failures such as implant breakage, loosening, fixation failure, instability, delayed soft tissue healing, soft tissue irritation, delayed union, non-union, may occur as a result of noncompliance to post-operative rehabilitation, excessive activities, or construct overloading.
- For safe effective use of the implant, the surgeon must be thoroughly familiar with the surgical technique, implant, and associated instruments.
- The devices in the system are not designed to withstand the stress of excessive weight bearing, load bearing, or physical activity.

- Improper application of any of the devices during implantation may also increase the possibility of loosening, or migration.
- DO NOT reuse any of the system's implantable components. Reuse may compromise the structural integrity of the construct.
- Protect the system's implantable components against scratching or nicking. Such stress concentration can lead to implant failure.
- Before using Skeletal Dynamic's medical devices, inspect all implants and instruments for wear, disfiguration and physical damage. If evidence of wear, disfiguration or physical damage is found, DO NOT use and contact your local Skeletal Dynamics representative or the Skeletal Dynamics Customer Care Department.
- DO NOT permanently implant the Skeletal Dynamics K-Wires; they are only intended to be used during provisional fixation.
- DO NOT permanently implant the pre-loaded Drill Guides or AIMing Guides; they are intended to be removed prior to screw insertion.
- DO NOT use pin/peg/screw lengths that will excessively protrude through the far cortex as it may result in soft tissue irritation.
- DO NOT mix system specific instrumentation from different systems or manufacturers for metallurgical, biomechanical and functional reasons.
- Dispose of contaminated implants and instruments per established facility guidelines and protocols.
- Caution should be taken for interference to pacemakers during electrocautery or by uncertified drill power sources.
- Seek medical help immediately if implant malfunctions.
- The benefits from implant surgery may not meet the patient's expectations or may deteriorate over time, requiring revision surgery to replace the implant or to carry out alternative procedures.
- When drilling into the proximal humerus, be sure to avoid drilling into the articular surfaces.
- Care should be taken that no implants are placed interfering with the articular surface.
- To maintain traceability of the system implantable components, record each of the respective components Lot numbers in the patient records.
- If the screws are not attached and/or fully tightened, a non-union, delayed union, soft tissue complication, or construct failure may occur.
- Use caution when drilling with K-Wires or drills as it may result in vascular or nerve injury.
- Ensure sufficient space is available for proper application when used in conjunction with other implants to prevent interference. Interference with other devices may lead to failure of the implant or postoperative complications.
- Caution should be taken when contouring plates. Bending the plates may weaken or break the plates.
- The maximum angulation of the variable angle locking screw should not exceed 10° from the trajectory of the respective hole.

Potential Adverse Events:

The following are potential risks associated with Proximal Humerus Fixation: discomfort or abnormal sensations, damage to nerves, vessels, or soft tissue, infection, edema or swelling, joint contractures, reduced or loss of range of motion (ROM), bone erosion, bone fracture, material sensitivity, stiffness, nonunion, persistent pain, disassembly, loosening or migration of the implants, undesirable shortening or lengthening of limb, implant failure, fretting and crevice corrosion, or damage to the articular surfaces.

MRI Safety Information:

The Proximal Humerus Fixation System implants have not been evaluated for safety and compatibility in the MRI environment. They have not been tested for heating, migration, or image artifact in the MRI environment. The safety of the Proximal Humerus Fixation System implants in the MRI environment is unknown. Scanning a patient who has this device may result in patient injury.

Directions for Use:

The Proximal Humerus implants and accessories should only be used by surgeons who have knowledge of the System. Each surgeon must evaluate the appropriateness for the use of the system based on their clinical experiences.

The surgeon should select the type and size implant to best meet the patient's needs. Although the surgeon is the medical intermediary between the company and the patient, this document contains important medical information that should be shared with the patient.

It is the responsibility of the surgeon to be familiar with the procedure before use of this device. Additionally, it is the responsibility of the surgeon to be familiar with relevant publications regarding the procedure prior to use. Please refer to the *Proximal Humerus Fixation System Surgical Technique Guide* (MKT-00485-00) to review the surgical approach as described by Jorge L. Orbay, M.D. of the *Miami Hand and Upper Extremity Institute* located in Miami, Florida.

Cleaning, Sterilization, and Inspection

For instructions on cleaning, disinfection, sterilization and inspection of the Proximal Humerus Fixation System products please refer to *Instructions for Reprocessing (Cleaning, Disinfection, and Sterilization) of Skeletal Dynamics Products* (IFU-04056-00). Refer to the *Proximal Humerus Plating System Surgical Technique Guide* (MKT-00485-00) for proper kitted tray arrangement.

Resources

The latest version of the Instructions for Use may be requested in a physical format by email (orders@skeletaldynamics.com) or by phone (+1-877-753-5396). The physical copy will be provided within 7 calendar days of receiving a request from the user or at the time of delivery of the device if so, requested at the time of order.

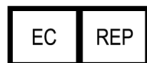
For the most current instructions for use visit www.skeletaldynamics.com/resources. Instructions for Use should always be reviewed before using or implanting a device.

Disclaimer of Warranty and Limited Remedies:

Skeletal Dynamics, Inc. makes no express or implied warranty, including any implied warranty of merchantability or fitness for a particular purpose, on the product(s) described in this publication. Skeletal Dynamics, Inc. shall not be liable under any circumstances for any direct, incidental or consequential damages other than as expressly provided by specific law. No person has authority to bind Skeletal Dynamics, Inc. to any representation or warranty except as specifically set forth in this publication. Descriptions or specifications provided by Skeletal Dynamics, Inc. in any publication are only included to generally describe the product when manufactured and do not constitute any express warranties.



Skeletal Dynamics, Inc
7300 N. Kendall Dr. / Suite 800
Miami, FL 33156
1-877-753-5396



European Representative:
Emergo Europe
Westervoortsedijk 60.
6827 AT Arnhem
The Netherlands



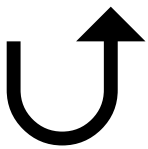
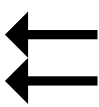
United Kingdom Representative
Emergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom



SYMBOLS GLOSSARY

Symbols that follow BS EN ISO 15223-1 / Medical Devices-Symbols to be used with medical device labels, labelling and information to be supplied.

Symbol	Symbol Reference Number and Title	Description
	5.2.7 Non-sterile	Indicates a medical device that has not been subjected to a sterilization process
	5.1.6 Catalogue number	Indicates the manufacturer's catalogue number so the medical device can be identified
	5.4.3 Consult instructions for use	Indicates the need for the user to consult the instructions for use
	5.1.5 Batch code	Indicates the manufacturer's batch code or lot can be identified
	5.4.2 Do not re-use	Indicates a medical device that is intended for one single use only
	5.1.3 Date of Manufacture	Indicates the date when the medical device was manufactured

	5.1.1 Manufacturer	Indicates the medical device manufacturer
	5.1.2 Authorized representative in the European Community	Indicates the authorized representative in the European union. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol
	Authorized Representative in the United Kingdom *Note symbol not found in BS EN ISO 15223-1	Indicates the authorized representative in the United Kingdom. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol
	5.2.8 Do not use if package is damaged	Indicates a medical device that should not be used if the package has been damaged or opened
	5.1.4 Use-by-date	Indicates the date after which the medical device is not to be used
	5.7.10 Unique device identifier	Indicates a carrier that contains unique device identifier information

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Proximal Humerus Plates	
FreeFix, Proximal Humerus Plate, 1 Slot, Right, Ti PHPF-1R (01)00841506138428  (01) 00841506138428	FreeFix, Proximal Humerus Plate, 1 Slot, Left, Ti PHPF-1L (01)00841506138435  (01) 00841506138435
FreeFix, Proximal Humerus Plate, 2 Slot, Right, Ti PHPF-2R (01)00841506138442  (01) 00841506138442	FreeFix, Proximal Humerus Plate, 2 Slot, Left, Ti PHPF-2L (01)00841506138459  (01) 00841506138459
FreeFix, Proximal Humerus Plate, 3 Slot, Right, Ti PHPF-3R (01)00841506145969  (01) 00841506145969	FreeFix, Proximal Humerus Plate, 3 Slot, Left, Ti PHPF-3L (01)00841506145976  (01) 00841506145976
FreeFix, Proximal Humerus Plate, Straight, 6 Slot, Right, Ti PHPF-S6R (01)00841506138466  (01) 00841506138466	FreeFix, Proximal Humerus Plate, Straight, 6 Slot, Left, Ti PHPF-S6L (01)00841506138473  (01) 00841506138473
FreeFix, Proximal Humerus Plate, Straight, 7 Slot, Right, Ti PHPF-S7R (01)00841506138480  (01) 00841506138480	FreeFix, Proximal Humerus Plate, Straight, 7 Slot, Left, Ti PHPF-S7L (01)00841506138497  (01) 00841506138497
FreeFix, Proximal Humerus Plate, Straight, 8 Slot, Right, Ti PHPF-S8R (01)00841506138503  (01) 00841506138503	FreeFix, Proximal Humerus Plate, Straight, 8 Slot, Left, Ti PHPF-S8L (01)00841506138510  (01) 00841506138510
FreeFix, Proximal Humerus Plate, Straight, 9 Slot, Right, Ti PHPF-S9R (01)00841506138527  (01) 00841506138527	FreeFix, Proximal Humerus Plate, Straight, 9 Slot, Left, Ti PHPF-S9L (01)00841506138534  (01) 00841506138534
FreeFix, Proximal Humerus Plate, Straight, 10 Slot, Right, Ti PHPF-S10R (01)00841506138541  (01) 00841506138541	FreeFix, Proximal Humerus Plate, Straight, 10 Slot, Left, Ti PHPF-S10L (01)00841506138558  (01) 00841506138558
FreeFix, Proximal Humerus Plate, Straight, 11 Slot, Right, Ti PHPF-S11R (01)00841506138565  (01) 00841506138565	FreeFix, Proximal Humerus Plate, Straight, 11 Slot, Left, Ti PHPF-S11L (01)00841506138572  (01) 00841506138572
FreeFix, Proximal Humerus Plate, Curved, 4 Slot, Right, Ti PHPF-C4R (01)00841506145884  (01) 00841506145884	FreeFix, Proximal Humerus Plate, Curved, 4 Slot, Left, Ti PHPF-C4L (01)00841506145891  (01) 00841506145891
FreeFix, Proximal Humerus Plate, Curved, 5 Slot, Right, Ti PHPF-C5R (01)00841506138589  (01) 00841506138589	FreeFix, Proximal Humerus Plate, Curved, 5 Slot, Left, Ti PHPF-C5L (01)00841506138596  (01) 00841506138596
FreeFix, Proximal Humerus Plate, Curved, 6 Slot, Right, Ti PHPF-C6R (01)00841506138602  (01) 00841506138602	FreeFix, Proximal Humerus Plate, Curved, 6 Slot, Left, Ti PHPF-C6L (01)00841506138619  (01) 00841506138619

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

FreeFix, Proximal Humerus Plate, Curved, 6 Slot Plus, Right, Ti PHPF-C6PR (01)00841506138626  (01)00841506138626	FreeFix, Proximal Humerus Plate, Curved, 6 Slot Plus, Left, Ti PHPF-C6PL (01)00841506138633  (01)00841506138633
FreeFix, Proximal Humerus Plate, Curved, 7 Slot, Right, Ti PHPF-C7R (01)00841506138640  (01)00841506138640	FreeFix, Proximal Humerus Plate, Curved, 7 Slot, Left, Ti PHPF-C7L (01)00841506138657  (01)00841506138657
FreeFix, Proximal Humerus Plate, Curved, 8 Slot, Right, Ti PHPF-C8R (01)00841506138664  (01)00841506138664	FreeFix, Proximal Humerus Plate, Curved, 8 Slot, Left, Ti PHPF-C8L (01)00841506138671  (01)00841506138671
FreeFix, Proximal Humerus Plate, Curved, 9 Slot, Right, Ti PHPF-C9R (01)00841506138688  (01)00841506138688	FreeFix, Proximal Humerus Plate, Curved, 9 Slot, Left, Ti PHPF-C9L (01)00841506138695  (01)00841506138695
Proximal Humerus, Buttress Plate, Ti PHP-BP (01)00841506138701  (01)00841506138701	Proximal Humerus, Suture Plate, Ti PHP-SP (01)00841506138718  (01)00841506138718
Proximal Humerus, AP Buttress Plate, Right, Ti PHP-BPAPR (01)00841506145907  (01)00841506145907	Proximal Humerus, AP Buttress Plate, Left, Ti PHP-BPAPL (01)00841506145914  (01)00841506145914
Proximal Humerus, Buttress Plate Screw, CoCr PHP-BPS (01)00841506138725  (01)00841506138725	
Smooth Peg, Locking, 6.0mm x XXmm, Ti	
Smooth Peg, Locking, 6.0mm x 30mm, Ti SPL-60300-TS (01)00841506138732  (01)00841506138732	Smooth Peg, Locking, 6.0mm x 48mm, Ti SPL-60480-TS (01)00841506138824  (01)00841506138824
Smooth Peg, Locking, 6.0mm x 32mm, Ti SPL-60320-TS (01)00841506138749  (01)00841506138749	Smooth Peg, Locking, 6.0mm x 50mm, Ti SPL-60500-TS (01)00841506138831  (01)00841506138831
Smooth Peg, Locking, 6.0mm x 34mm, Ti SPL-60340-TS (01)00841506138756  (01)00841506138756	Smooth Peg, Locking, 6.0mm x 52mm, Ti SPL-60520-TS (01)00841506138848  (01)00841506138848
Smooth Peg, Locking, 6.0mm x 36mm, Ti SPL-60360-TS (01)00841506138763  (01)00841506138763	Smooth Peg, Locking, 6.0mm x 54mm, Ti SPL-60540-TS (01)00841506138855  (01)00841506138855
Smooth Peg, Locking, 6.0mm x 38mm, Ti SPL-60380-TS (01)00841506138770  (01)00841506138770	Smooth Peg, Locking, 6.0mm x 56mm, Ti SPL-60560-TS (01)00841506138862  (01)00841506138862
Smooth Peg, Locking, 6.0mm x 40mm, Ti SPL-60400-TS (01)00841506138787  (01)00841506138787	Smooth Peg, Locking, 6.0mm x 58mm, Ti SPL-60580-TS (01)00841506138879  (01)00841506138879

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Smooth Peg, Locking, 6.0mm x 42mm, Ti SPL-60420-TS (01)00841506138794	 (01) 00841506138794	Smooth Peg, Locking, 6.0mm x 60mm, Ti SPL-60600-TS (01)00841506138886	 (01) 00841506138886
Smooth Peg, Locking, 6.0mm x 44mm, Ti SPL-60440-TS (01)00841506138800	 (01) 00841506138800	Smooth Peg, Locking, 6.0mm x 65mm, Ti SPL-60650-TS (01)00841506138893	 (01) 00841506138893
Smooth Peg, Locking, 6.0mm x 46mm, Ti SPL-60460-TS (01)00841506138817	 (01) 00841506138817	Smooth Peg, Locking, 6.0mm x 70mm, Ti SPL-60700-TS (01)00841506138909	 (01) 00841506138909
Smooth Peg, Locking, 3.5mm x XXmm, Ti			
Smooth Peg, Locking, 3.5mm x 20mm, Ti SPL-35200-TS (01)00841506138992	 (01) 00841506138992	Smooth Peg, Locking, 3.5mm x 44mm, Ti SPL-35440-TS (01)00841506139111	 (01) 00841506139111
Smooth Peg, Locking, 3.5mm x 22mm, Ti SPL-35220-TS (01)00841506139005	 (01) 00841506139005	Smooth Peg, Locking, 3.5mm x 46mm, Ti SPL-35460-TS (01)00841506139128	 (01) 00841506139128
Smooth Peg, Locking, 3.5mm x 24mm, Ti SPL-35240-TS (01)00841506139012	 (01) 00841506139012	Smooth Peg, Locking, 3.5mm x 48mm, Ti SPL-35480-TS (01)00841506139135	 (01) 00841506139135
Smooth Peg, Locking, 3.5mm x 26mm, Ti SPL-35260-TS (01)00841506139029	 (01) 00841506139029	Smooth Peg, Locking, 3.5mm x 50mm, Ti SPL-35500-TS (01)00841506139142	 (01) 00841506139142
Smooth Peg, Locking, 3.5mm x 28mm, Ti SPL-35280-TS (01)00841506139036	 (01) 00841506139036	Smooth Peg, Locking, 3.5mm x 52mm, Ti SPL-35520-TS (01)00841506139159	 (01) 00841506139159
Smooth Peg, Locking, 3.5mm x 30mm, Ti SPL-35300-TS (01)00841506139043	 (01) 00841506139043	Smooth Peg, Locking, 3.5mm x 54mm, Ti SPL-35540-TS (01)00841506139166	 (01) 00841506139166
Smooth Peg, Locking, 3.5mm x 32mm, Ti SPL-35320-TS (01)00841506139050	 (01) 00841506139050	Smooth Peg, Locking, 3.5mm x 56mm, Ti SPL-35560-TS (01)00841506139173	 (01) 00841506139173
Smooth Peg, Locking, 3.5mm x 34mm, Ti SPL-35340-TS (01)00841506139067	 (01) 00841506139067	Smooth Peg, Locking, 3.5mm x 58mm, Ti SPL-35580-TS (01)00841506139180	 (01) 00841506139180
Smooth Peg, Locking, 3.5mm x 36mm, Ti SPL-35360-TS (01)00841506139074	 (01) 00841506139074	Smooth Peg, Locking, 3.5mm x 60mm, Ti SPL-35600-TS (01)00841506139197	 (01) 00841506139197
Smooth Peg, Locking, 3.5mm x 38mm, Ti SPL-35380-TS (01)00841506139081	 (01) 00841506139081	Smooth Peg, Locking, 3.5mm x 65mm, Ti SPL-35650-TS (01)00841506139203	 (01) 00841506139203

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Smooth Peg, Locking, 3.5mm x 40mm, Ti SPL-35400-TS (01)00841506139098  (01) 00841506139098	Smooth Peg, Locking, 3.5mm x 70mm, Ti SPL-35700-TS (01)00841506139210  (01) 00841506139210
Smooth Peg, Locking, 3.5mm x 42mm, Ti SPL-35420-TS (01)00841506139104  (01) 00841506139104	
Screw, Metaphyseal Locking, 3.7mm x XXmm, Ti	
Screw, Metaphyseal Locking, 3.7mm x 20mm, Ti MTL-37200-TS (01)00841506139227  (01) 00841506139227	Screw, Metaphyseal Locking, 3.7mm x 44mm, Ti MTL-37440-TS (01)00841506139340  (01) 00841506139340
Screw, Metaphyseal Locking, 3.7mm x 22mm, Ti MTL-37220-TS (01)00841506139234  (01) 00841506139234	Screw, Metaphyseal Locking, 3.7mm x 46mm, Ti MTL-37460-TS (01)00841506139357  (01) 00841506139357
Screw, Metaphyseal Locking, 3.7mm x 24mm, Ti MTL-37240-TS (01)00841506139241  (01) 00841506139241	Screw, Metaphyseal Locking, 3.7mm x 48mm, Ti MTL-37480-TS (01)00841506139364  (01) 00841506139364
Screw, Metaphyseal Locking, 3.7mm x 26mm, Ti MTL-37260-TS (01)00841506139258  (01) 00841506139258	Screw, Metaphyseal Locking, 3.7mm x 50mm, Ti MTL-37500-TS (01)00841506139371  (01) 00841506139371
Screw, Metaphyseal Locking, 3.7mm x 28mm, Ti MTL-37280-TS (01)00841506139265  (01) 00841506139265	Screw, Metaphyseal Locking, 3.7mm x 52mm, Ti MTL-37520-TS (01)00841506139388  (01) 00841506139388
Screw, Metaphyseal Locking, 3.7mm x 30mm, Ti MTL-37300-TS (01)00841506139272  (01) 00841506139272	Screw, Metaphyseal Locking, 3.7mm x 54mm, Ti MTL-37540-TS (01)00841506139395  (01) 00841506139395
Screw, Metaphyseal Locking, 3.7mm x 32mm, Ti MTL-37320-TS (01)00841506139289  (01) 00841506139289	Screw, Metaphyseal Locking, 3.7mm x 56mm, Ti MTL-37560-TS (01)00841506139401  (01) 00841506139401
Screw, Metaphyseal Locking, 3.7mm x 34mm, Ti MTL-37340-TS (01)00841506139296  (01) 00841506139296	Screw, Metaphyseal Locking, 3.7mm x 58mm, Ti MTL-37580-TS (01)00841506139418  (01) 00841506139418
Screw, Metaphyseal Locking, 3.7mm x 36mm, Ti MTL-37360-TS (01)00841506139302  (01) 00841506139302	Screw, Metaphyseal Locking, 3.7mm x 60mm, Ti MTL-37600-TS (01)00841506139425  (01) 00841506139425
Screw, Metaphyseal Locking, 3.7mm x 38mm, Ti MTL-37380-TS (01)00841506139319  (01) 00841506139319	Screw, Metaphyseal Locking, 3.7mm x 65mm, Ti MTL-37650-TS (01)00841506139432  (01) 00841506139432
Screw, Metaphyseal Locking, 3.7mm x 40mm, Ti MTL-37400-TS (01)00841506139326  (01) 00841506139326	Screw, Metaphyseal Locking, 3.7mm x 70mm, Ti MTL-37700-TS (01)00841506139449  (01) 00841506139449

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Screw, Metaphyseal Locking, 3.7mm x 42mm, Ti MTLS-37420-TS (01)00841506139333  (01) 00841506139333	
Screw, Metaphyseal, Compression, 3.7mm x XXmm, Ti	
Screw, Metaphyseal, Compression, 3.7mm x 20mm, Ti MTC-37200-TS (01)00841506139456  (01) 00841506139456	Screw, Metaphyseal, Compression, 3.7mm x 44mm, Ti MTC-37440-TS (01)00841506139579  (01) 00841506139579
Screw, Metaphyseal, Compression, 3.7mm x 22mm, Ti MTC-37220-TS (01)00841506139463  (01) 00841506139463	Screw, Metaphyseal, Compression, 3.7mm x 46mm, Ti MTC-37460-TS (01)00841506139586  (01) 00841506139586
Screw, Metaphyseal, Compression, 3.7mm x 24mm, Ti MTC-37240-TS (01)00841506139470  (01) 00841506139470	Screw, Metaphyseal, Compression, 3.7mm x 48mm, Ti MTC-37480-TS (01)00841506139593  (01) 00841506139593
Screw, Metaphyseal, Compression, 3.7mm x 26mm, Ti MTC-37260-TS (01)00841506139487  (01) 00841506139487	Screw, Metaphyseal, Compression, 3.7mm x 50mm, Ti MTC-37500-TS (01)00841506139609  (01) 00841506139609
Screw, Metaphyseal, Compression, 3.7mm x 28mm, Ti MTC-37280-TS (01)00841506139494  (01) 00841506139494	Screw, Metaphyseal, Compression, 3.7mm x 52mm, Ti MTC-37520-TS (01)00841506139616  (01) 00841506139616
Screw, Metaphyseal, Compression, 3.7mm x 30mm, Ti MTC-37300-TS (01)00841506139500  (01) 00841506139500	Screw, Metaphyseal, Compression, 3.7mm x 54mm, Ti MTC-37540-TS (01)00841506139623  (01) 00841506139623
Screw, Metaphyseal, Compression, 3.7mm x 32mm, Ti MTC-37320-TS (01)00841506139517  (01) 00841506139517	Screw, Metaphyseal, Compression, 3.7mm x 56mm, Ti MTC-37560-TS (01)00841506139630  (01) 00841506139630
Screw, Metaphyseal, Compression, 3.7mm x 34mm, Ti MTC-37340-TS (01)00841506139524  (01) 00841506139524	Screw, Metaphyseal, Compression, 3.7mm x 58mm, Ti MTC-37580-TS (01)00841506139647  (01) 00841506139647
Screw, Metaphyseal, Compression, 3.7mm x 36mm, Ti MTC-40360-TS (01)00841506139531  (01) 00841506139531	Screw, Metaphyseal, Compression, 3.7mm x 60mm, Ti MTC-37600-TS (01)00841506139654  (01) 00841506139654
Screw, Metaphyseal, Compression, 3.7mm x 38mm, Ti MTC-37380-TS (01)00841506139548  (01) 00841506139548	Screw, Metaphyseal, Compression, 3.7mm x 65mm, Ti MTC-37650-TS (01)00841506139661  (01) 00841506139661
Screw, Metaphyseal, Compression, 3.7mm x 40mm, Ti MTC-37400-TS (01)00841506139555  (01) 00841506139555	Screw, Metaphyseal, Compression, 3.7mm x 70mm, Ti MTC-37700-TS (01)00841506139678  (01) 00841506139678
Screw, Metaphyseal, Compression, 3.7mm x 42mm, Ti MTC-37420-TS (01)00841506139562  (01) 00841506139562	

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Screw, Variable Angle Locking, 3.5mm x XXmm, CoCr

Screw, Variable Angle Locking, 3.5mm x 20mm, CoCr VAL-35200-CC (01)00841506139760  (01) 00841506139760	Screw, Variable Angle Locking, 3.5mm x 44mm, CoCr VAL-35440-CC (01)00841506139883  (01) 00841506139883
Screw, Variable Angle Locking, 3.5mm x 22mm, CoCr VAL-35220-CC (01)00841506139777  (01) 00841506139777	Screw, Variable Angle Locking, 3.5mm x 46mm, CoCr VAL-35460-CC (01)00841506139890  (01) 00841506139890
Screw, Variable Angle Locking, 3.5mm x 24mm, CoCr VAL-35240-CC (01)00841506139784  (01) 00841506139784	Screw, Variable Angle Locking, 3.5mm x 48mm, CoCr VAL-35480-CC (01)00841506139906  (01) 00841506139906
Screw, Variable Angle Locking, 3.5mm x 26mm, CoCr VAL-35260-CC (01)00841506139791  (01) 00841506139791	Screw, Variable Angle Locking, 3.5mm x 50mm, CoCr VAL-35500-CC (01)00841506139913  (01) 00841506139913
Screw, Variable Angle Locking, 3.5mm x 28mm, CoCr VAL-35280-CC (01)00841506139807  (01) 00841506139807	Screw, Variable Angle Locking, 3.5mm x 52mm, CoCr VAL-35520-CC (01)00841506139920  (01) 00841506139920
Screw, Variable Angle Locking, 3.5mm x 30mm, CoCr VAL-35300-CC (01)00841506139814  (01) 00841506139814	Screw, Variable Angle Locking, 3.5mm x 54mm, CoCr VAL-35540-CC (01)00841506139937  (01) 00841506139937
Screw, Variable Angle Locking, 3.5mm x 32mm, CoCr VAL-35320-CC (01)00841506139821  (01) 00841506139821	Screw, Variable Angle Locking, 3.5mm x 56mm, CoCr VAL-35560-CC (01)00841506139944  (01) 00841506139944
Screw, Variable Angle Locking, 3.5mm x 34mm, CoCr VAL-35340-CC (01)00841506139838  (01) 00841506139838	Screw, Variable Angle Locking, 3.5mm x 58mm, CoCr VAL-35580-CC (01)00841506139951  (01) 00841506139951
Screw, Variable Angle Locking, 3.5mm x 36mm, CoCr VAL-35360-CC (01)00841506139845  (01) 00841506139845	Screw, Variable Angle Locking, 3.5mm x 60mm, CoCr VAL-35600-CC (01)00841506139968  (01) 00841506139968
Screw, Variable Angle Locking, 3.5mm x 38mm, CoCr VAL-35380-CC (01)00841506139852  (01) 00841506139852	Screw, Variable Angle Locking, 3.5mm x 65mm, CoCr VAL-35650-CC (01)00841506139975  (01) 00841506139975
Screw, Variable Angle Locking, 3.5mm x 40mm, CoCr VAL-35400-CC (01)00841506139869  (01) 00841506139869	Screw, Variable Angle Locking, 3.5mm x 70mm, CoCr VAL-35700-CC (01)00841506139982  (01) 00841506139982
Screw, Variable Angle Locking, 3.5mm x 42mm, CoCr VAL-35420-CC (01)00841506139876  (01) 00841506139876	

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Screw, Variable Angle Locking, 3.0mm x XXmm, CoCr

Screw, Variable Angle Locking, 3.0mm x 20mm CoCr VAL-30200-CC (01)00841506146126  (01) 00841506146126	Screw, Variable Angle Locking, 3.0mm x 36mm, CoCr VAL-30360-CC (01)00841506146201  (01) 00841506146201
Screw, Variable Angle Locking, 3.0mm x 22mm CoCr VAL-30220-CC (01)00841506146133  (01) 00841506146133	Screw, Variable Angle Locking, 3.0mm x 38mm, CoCr VAL-30380-CC (01)00841506146218  (01) 00841506146218
Screw, Variable Angle Locking, 3.0mm x 24mm, CoCr VAL-30240-CC (01)00841506146140  (01) 00841506146140	Screw, Variable Angle Locking, 3.0mm x 40mm, CoCr VAL-30400-CC (01)00841506146225  (01) 00841506146225
Screw, Variable Angle Locking, 3.0mm x 26mm, CoCr VAL-30260-CC (01)00841506146157  (01) 00841506146157	Screw, Variable Angle Locking, 3.0mm x 42mm, CoCr VAL-30420-CC (01)00841506146232  (01) 00841506146232
Screw, Variable Angle Locking, 3.0mm x 28mm, CoCr VAL-30280-CC (01)00841506146164  (01) 00841506146164	Screw, Variable Angle Locking, 3.0mm x 44mm, CoCr VAL-30440-CC (01)00841506146249  (01) 00841506146249
Screw, Variable Angle Locking, 3.0mm x 30mm, CoCr VAL-30300-CC (01)00841506146171  (01) 00841506146171	Screw, Variable Angle Locking, 3.0mm x 46mm, CoCr VAL-30460-CC (01)00841506146263  (01) 00841506146263
Screw, Variable Angle Locking, 3.0mm x 32mm, CoCr VAL-30320-CC (01)00841506146188  (01) 00841506146188	Screw, Variable Angle Locking, 3.0mm x 48mm, CoCr VAL-30480-CC (01)00841506146270  (01) 00841506146270
Screw, Variable Angle Locking, 3.0mm x 34mm, CoCr VAL-30340-CC (01)00841506146195  (01) 00841506146195	Screw, Variable Angle Locking, 3.0mm x 50mm, CoCr VAL-30500-CC (01)00841506146287  (01) 00841506146287

Screw, FreeFix Locking, 4.5mm x XXmm, Ti

Screw, FreeFix Locking, 4.5mm x 14mm, Ti FFL-45140-TS (01)00841506129655  (01) 00841506129655	Screw, FreeFix Locking, 4.5mm x 26mm, Ti FFL-45260-TS (01)00841506129716  (01) 00841506129716
Screw, FreeFix Locking, 4.5mm x 16mm, Ti FFL-45160-TS (01)00841506129662  (01) 00841506129662	Screw, FreeFix Locking, 4.5mm x 28mm, Ti FFL-45280-TS (01)00841506129723  (01) 00841506129723
Screw, FreeFix Locking, 4.5mm x 18mm, Ti FFL-45180-TS (01)00841506129679  (01) 00841506129679	Screw, FreeFix Locking, 4.5mm x 30mm, Ti FFL-45300-TS (01)00841506129730  (01) 00841506129730
Screw, FreeFix Locking, 4.5mm x 20mm, Ti FFL-45200-TS (01)00841506129686  (01) 00841506129686	Screw, FreeFix Locking, 4.5mm x 32mm, Ti FFL-45320-TS (01)00841506129747  (01) 00841506129747

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Screw, FreeFix Locking, 4.5mm x 22mm, Ti FFL-45220-TS (01)00841506129693  (01)00841506129693	Screw, FreeFix Locking, 4.5mm x 34mm, Ti FFL-45340-TS (01)00841506129754  (01)00841506129754
Screw, FreeFix Locking, 4.5mm x 24mm, Ti FFL-45240-TS (01)00841506129709  (01)00841506129709	
Screw, FreeFix Compression, 4.5mm x XXmm, Ti	
Screw, FreeFix Compression, 4.5mm x 14mm, Ti FFC-45140-TS (01)00841506129822  (01)00841506129822	Screw, FreeFix Compression, 4.5mm x 28mm, Ti FFC-45280-TS (01)00841506129891  (01)00841506129891
Screw, FreeFix Compression, 4.5mm x 16mm, Ti FFC-45160-TS (01)00841506129839  (01)00841506129839	Screw, FreeFix Compression, 4.5mm x 30mm, Ti FFC-45300-TS (01)00841506129907  (01)00841506129907
Screw, FreeFix Compression, 4.5mm x 18mm, Ti FFC-45180-TS (01)00841506129846  (01)00841506129846	Screw, FreeFix Compression, 4.5mm x 32mm, Ti FFC-45320-TS (01)00841506129914  (01)00841506129914
Screw, FreeFix Compression, 4.5mm x 20mm, Ti FFC-45200-TS (01)00841506129853  (01)00841506129853	Screw, FreeFix Compression, 4.5mm x 34mm, Ti FFC-45340-TS (01)00841506129921  (01)00841506129921
Screw, FreeFix Compression, 4.5mm x 22mm, Ti FFC-45220-TS (01)00841506129860  (01)00841506129860	Screw, FreeFix Compression, 4.5mm x 36mm, Ti FFC-45360-TS (01)00841506129938  (01)00841506129938
Screw, FreeFix Compression, 4.5mm x 24mm, Ti FFC-45240-TS (01)00841506129877  (01)00841506129877	Screw, FreeFix Compression, 4.5mm x 38mm, Ti FFC-45380-TS (01)00841506129945  (01)00841506129945
Screw, FreeFix Compression, 4.5mm x 26mm, Ti FFC-45260-TS (01)00841506129884  (01)00841506129884	Screw, FreeFix Compression, 4.5mm x 40mm, Ti FFC-45400-TS (01)00841506129952  (01)00841506129952
Screw, FreeFix Compression, 3.5mm x XXmm, Ti	
Screw, FreeFix Compression, 3.5mm x 14mm, Ti FFC-35140-TS (01)00841506129976  (01)00841506129976	Screw, FreeFix Compression, 3.5mm x 28mm, Ti FFC-35280-TS (01)00841506130361  (01)00841506130361
Screw, FreeFix Compression, 3.5mm x 16mm, Ti FFC-35160-TS (01)00841506130293  (01)00841506130293	Screw, FreeFix Compression, 3.5mm x 30mm, Ti FFC-35300-TS (01)00841506130378  (01)00841506130378
Screw, FreeFix Compression, 3.5mm x 18mm, Ti FFC-35180-TS (01)00841506130316  (01)00841506130316	Screw, FreeFix Compression, 3.5mm x 32mm, Ti FFC-35320-TS (01)00841506130385  (01)00841506130385

Skeletal Dynamics Proximal Humerus Plating System

Inventory Control Sheet

Screw, FreeFix Compression, 3.5mm x 20mm, Ti FFC-35200-TS (01)00841506130323  (01) 00841506130323	Screw, FreeFix Compression, 3.5mm x 34mm, Ti FFC-35340-TS (01)00841506130392  (01) 00841506130392
Screw, FreeFix Compression, 3.5mm x 22mm, Ti FFC-35220-TS (01)00841506130330  (01) 00841506130330	Screw, FreeFix Compression, 3.5mm x 36mm, Ti FFC-35360-TS (01)00841506130408  (01) 00841506130408
Screw, FreeFix Compression, 3.5mm x 24mm, Ti FFC-35240-TS (01)00841506130347  (01) 00841506130347	Screw, FreeFix Compression, 3.5mm x 38mm, Ti FFC-35380-TS (01)00841506130415  (01) 00841506130415
Screw, FreeFix Compression, 3.5mm x 26mm, Ti FFC-35260-TS (01)00841506130354  (01) 00841506130354	Screw, FreeFix Compression, 3.5mm x 40mm, Ti FFC-35400-TS (01)00841506130422  (01) 00841506130422
Steinmann Pin & K-Wire	
Steinmann Pin, Single Diamond, 2.4mm x 229mm SPIN-SD-24229 (01)00841506138916  (01) 00841506138916	2.4mm Steinmann Pin Probe PRB-24 (01)00841506138923  (01) 00841506138923
K-Wire, Single Diamond Tip, Threaded with Stop, 2.0mm x 152mm KWIR-DTS-20152 (01)00841506146393  (01) 00841506146393	K-Wire, Standard Tip, HPS PLS, 1.1 mm x 152 mm KWIR-HPS-PLS (01)00841506117812  (01) 00841506117812